

# **3.5MM LIFT T PLATING SYSTEM SURGICAL TECHNIQUE**

<b>Doc. No.</b>	<b>Issue No/Rev No</b>	<b>Dated</b>	<b>Page No</b>
YIPL/ST/24	01/01	27/01/2020	Page 1 of 21

## Index

---


1. Description of Surgical Technique .....	3
2. Feature & Benefits.....	4
3. AO Principle .....	5
4. Indications.....	6
5. Intended Purpose .....	7
6. Contraindications.....	8
7. Surgical Steps .....	9
8. Implant Removal .....	12
9. Caution .....	13
10. Disposal of Implants .....	14
11. Packaging Material Disposal .....	15
12. MRI Information .....	16
13. Note .....	17

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 2 of 21

## 1. Description of Surgical Technique

---

**Name** - 3.5MM LIFT T PLATING SYSTEM

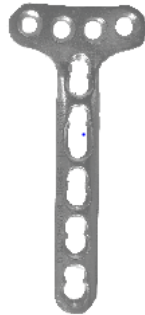

The 3.5MM Lift T plating system, featuring locking compression technology, is indicated for intra- and extra-articular fractures and osteotomies of the distal radius.

Available in SS316L & Titanium Gr. 5 materials.


Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 3 of 21

## 2. Feature & Benefits


---

- Anatomically precontoured
- Minimal irritation of ligaments and soft tissue from a flat plate and screw profile, rounded edges and polished surfaces.
- Depending on the indication, plates are selected with juxta articular or extra articular placement. Both locking, Cancellous locking and Cortical screws 3.5mm can be inserted in the shaft.

### T Plate Right Angled 3 Hole

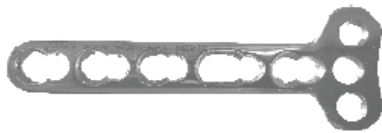

### T Plate Right Angled 4 Hole

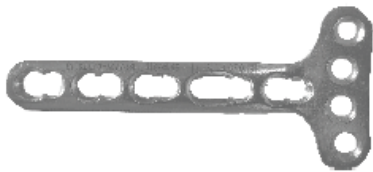

### Oblique T Plate



Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 4 of 21

### 3. AO Principle

---

In 1958, the AO formulated four basic principles, which have become the guidelines for internal fixation:

1. Anatomic reduction: Fracture Reduction and fixation to restore anatomical relationships.
2. Stable fixation: Fracture fixation providing absolute or relative stability, as required by the patient, the injury, and the personality of the fracture.
3. Preservation of blood supply: Preservation of the blood supply to soft tissues and bone by gentle reduction techniques and careful handling.
4. Early, active mobilization: Early and Safe Mobilization and rehabilitation of the injured part and the patient as a whole.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 5 of 21

## 4. Indications

---

- Displaced extra-articular and intra-articular distal radius fractures and corrective osteotomies of the distal radius. volar approach.
- Extra-articular fractures with metaphyseal defect (AO classification 23-A3)
- Open joint reconstruction (AO classification 23-C1, C2, C3)
- Combination of distal radius with carpal and metacarpal fractures
- Corrective osteotomies

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 6 of 21

## 5. Intended Purpose

---

The 3.5MM Lift T plating system is intended for temporary fixation, correction or stabilization in the Distal Radius anatomical region.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 7 of 21

## 6. Contraindications

---

- Since external fixation devices are often used in emergency situations to treat patients with acute injuries, there are no absolute contraindications for use. The surgeon's education, training and professional judgment must be relied upon to choose the most appropriate device and treatment for each individual patient. Whenever possible, the device chosen should be of a type indicated for the fracture being treated and/or for the procedure being utilized.
- Insufficient quantity or quality of bone which would inhibit appropriate fixation of the device.
- Compromised vascularity that would inhibit adequate blood supply to the fracture or operative site.
- Previous history of infections.
- Any neuromuscular deficit which could interfere with the patient's ability to limit weight bearing.
- Any neuromuscular deficit which places an unusually heavy load on the device during the healing period.
- Malignancy in the fracture area.
- Mental, physical or neurological conditions which may impair the patient's
- Ability to cooperate with the post-operative regimen.
- Patients with a compromised immune system.
- Pre-existing internal fixation that prohibits proper pin placement

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 8 of 21



## 7. Surgical Steps

---

- **Preoperative Planning- General Technique**

### Plate selection and contouring

The plates are available in various shapes and lengths which allow the surgeon to select fragment-specific treatment of distal radius fractures. Decide on the desired volar approach and select the plates according to the fracture pattern and anatomy of the radius.

### Precautions:

- The plate holes have been designed to accept some degree of deformation. The undercuts help ensure that the threaded holes will not be distorted with typical contouring. Significant distortion of the threaded holes will re-duce locking effectiveness.
- Reverse bending or use of the incorrect instrumentation for bending may weaken the plate and lead to premature plate failure (e.g. breakage). Do not bend the plate beyond what is required to match the anatomy.
- Temporary fixation of fracture with K- wire



- Reduction can be preliminarily held with K-wires of 1.5mm or 2 mm. A wire introduced across the radial styloid will fit into a small notch (horse-shoe tip) in the distal end of the straight plate.

### Screw insertion

Determine whether cortical screws or locking screws will be used for Fixation in the shaft. Locking screws in the distal arm (head of the plate) may be an advantage to support the articular surface and prevent loss of reduction.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 9 of 21

**Recommendation:** Use locking head screws in the distal arm of the plates, and locking head and/or cortical screws in the shaft of the plates. If a combination of cortical screws and locking head screws is used, a cortical screw should be used first to pull the plate to the bone.

**Warning:** If a locking head screw is used first, care should be taken to ensure that the plate is held securely to the bone, to avoid spinning of the plate.

**Insertion of Cortical Screws:**



**Pre-drill screw hole**

The insertion of cortical screws – 3.5 mm is described using the example of a T plate. According to the selected screw diameter use the appropriate drill guide & tap guide 2.5mm/3.5mm to pre-drill the screw hole either neutrally (buttress) or off-center (compression). For the Cortical screw 3.5 mm, use the 2.5 mm drill bit for the gliding hole.

**Determine screw length**

Use the Depth Gauge for screws 3.5 mm to determine the screw length.

**Pick up screw**

Select and pick up the appropriate cortical screw using the Hexagonal Screwdriver 3.5mm.

Insert self-tapping Cortical Screw- 3.5mm



Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 10 of 21

Insert the self-tapping cortical screw with the Hexagonal Screw Driver 3.5mm.

**Insertion of Locking Screws -3.5mm:**

Insert Lift drill sleeve 2.8 mm



The insertion of locking screws is described using the example of an T plate. Screw the Locking Drill Sleeve for 3.5mm screws vertically into a threaded hole until fully seated. This is very much needed to ensure central drilling of the screw hole so that the final screw head shall fit into the plate’s hole flushing to the plate and locking threads of the screws shall engaged in plate firmly.

**Predrill screw hole**

With the Drill Sleeve for 3.5mm screws, use 2.8mm drill bit to drill to the desired depth.

**Determine screw length**

Use the Depth Gauge for screws 3.5mm to determine the screw length.

**Pick up screw**

Select and pick up the appropriate screw using the Hexagonal Screw Driver 3.5mm



**Insert Locking screw 3.5mm.**



Insert the locking screw manually with the Hexagonal Screwdriver. Carefully tighten the locking screw, as excessive force is not necessary to produce effective screw locking. Alternatively, to apply the correct amount of torque use the Torque Limit Screw Driver- 3.5mm for torque up to 1.5Nm at the time of inserting final few threads of the screw.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 11 of 21

## 8. Implant Removal

---

The 3.5mm LIFT T Plates should first be removed by following screw removal technique of cortical screws. Take care while removing locking screws, first unlock all screws from the plate with Hexagonal Screw Driver 3.5 mm then remove the screws completely from the bone. This prevents rotation of the plate when removing the last locking screw. Ensure that the tip of the screw driver sits fully into the head of the screws. Partial engagement may lead to wear out of screw head or screw driver tip. Don't use high torque while removing the screws. If screw head gets damaged during removal, use the screw removal instruments to remove damage head screws.

Note: The final decision of removing the implants shall be taken by the operating surgeon only. It is recommended that the implant used as an aid for healing should be removed once its service is over after proper consultation and examination by the operating surgeon in final follow up, particularly in younger and more active patients.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 12 of 21

## 9. Caution

---

### Used Implants:

Used implants which appear un-damaged may have internal and/or external defects. It is possible that individual stress analysis of each part fail to reveal the accumulated stress on the metals as a result of use within the body. This may lead ultimately to implant failure after certain point of time due to metal fatigue. Therefore, reuse of implants is strictly not recommended.

**SINGLE BRAND USAGE:** Implant components from one manufacture should not be used with those of another. Implants from each manufacture may have metal, dimensions and design differences so that the use in conjunction with different brands of devices may lead to inadequate fixation or adverse performances of the devices.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 13 of 21

## **10. Disposal of Implants**

---

Every used or removed implant must be discarded after use and must never be re- used. It should be bent or scratched & then disposed of properly so that it becomes unfit for reuse. While disposing it off, it should be ensured that the discarded implant does not pose any threat to children, stray animals and environment. Dispose of the implants as per applicable medical practices and local, state and country specific regulatory requirement of Bio Medical Waste rules.

<b>Doc. No.</b>	<b>Issue No/Rev No</b>	<b>Dated</b>	<b>Page No</b>
YIPL/ST/24	01/01	27/01/2020	Page 14 of 21

## **11. Packaging Material Disposal**

---

The packaging material of this device is made special packing material and therefore if swallowed, may cause choking Hazards. Therefore, it should be disposed of in such a way that keep out of reach of children and stray animals.

<b>Doc. No.</b>	<b>Issue No/Rev No</b>	<b>Dated</b>	<b>Page No</b>
YIPL/ST/24	01/01	27/01/2020	Page 15 of 21

## 12. MRI Information

---

### IMPORTANT:

- Yogeshwar Implants (I) Pvt. Ltd. implants are manufactured from SS 316L and Titanium Grade 5 material both are non-magnetic material, hence it do not pose any safety risk.
- Patients should be directed to seek a medical opinion before entering potentially adverse environments that could affect the performance of the implants, such as electromagnetic or magnetic field or including a magnetic resonance environment.
- Doctor shall conduct a Risk Benefit Analysis before directing the patient to enter electromagnetic or magnetic fields or including a magnetic resonance environment.
- The Yogeshwar Implants (I) Pvt. Ltd. implants has not been evaluated for safety and compatibility in the MR environment but on the basis of literature study below mentioned points can be taken care during MRI
  - ✓ The minimum recommended time after the implantation that allows patients to safely undergo MRI examination or allowing the patient or an individual to enter the MRI environment is 6 (six) weeks.
  - ✓ The maximum recommended time limit for MRI examination in patients implanted with the evaluated device is 30 min with a scanner operating at 1.5T (Tesla) or less.

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 16 of 21



## 13. Note

### T Plate Right Angled 3 Hole

Cat No. For SS	0206
Cat No. For Tit.	1306
Available In	Universal
No. of Shaft Holes	3 to 9
Locking Screw	3,5 mm
Cortical Screw	3,5 mm
Cancellous Screw	3,5 mm

**Profile**

Thickness : 2,0 mm  
Width : 10,0 mm  
Fixation with 3,5 mm screw.

**Indication**

for extra- and intra-articular osteosynthesis of fractures of the distal radius volar side

**Features**

Good for the stabilisation of displaced distal radius fractures

**Use**

Fracture for Distal Radius Bone.



### T Plate Right Angled 4 Hole

Cat No. For SS	0207
Cat No. For Tit.	1307
Available In	Universal
No. of Shaft Holes	3 to 9
Locking Screw	3,5 mm
Cortical Screw	3,5 mm
Cancellous Screw	3,5 mm

**Profile**

Thickness : 2,0 mm  
Width : 10,0 mm  
Fixation with 3,5 mm screw.

**Indication**

For extra- and intra-articular osteosynthesis of fractures of the distal radius volar side.

**Features**

Good for the stabilisation of displaced distal radius fractures

**Use**

Fracture for Distal Radius Bone.



Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 17 of 21

## PLATES FOR 3.5 MM SCREWS

### Oblique T Plate

Cat No. For SS	0205
Cat No. For Ti	1305
Available In	R/L
No. of Shaft Holes	3 to 9
Locking Screw	3.5 mm
Cortical Screw	3.5 mm
Cancellous Screw	3.5 mm

**Profile**

Thickness : 2.0 mm  
Width : 10.0 mm  
Fixation with 3.5 mm screw.

**Indication**

for extra- and intra-articular osteosynthesis of fractures of the distal radius volar side

**Features**

Good for the stabilisation of displaced distal radius fractures

**Use**

Fracture for Distal Radius Bone,



**LIFT HEX. CORTICAL SCREW-3.5 MM 20 TPI SELF TAPPING** 1110  
**LIFT STAR CORTICAL SCREW-3.5 MM 20 TPI SELF TAPPING** 0810



10MM TO 40MM ( AT THE DIFFERENCE OF 2MM)

**LIFT HEX. LOCKING SCREWS 3.5MM - 10MM TO 50MM** 1127  
**LIFT STAR LOCKING SCREWS 3.5MM - 10MM TO 50MM** 0837



( DIFF. OF 2MM UPTO 40MM & AFTER THAT 5MM UPTO 85MM)

**LIFT HEX. CANCELLOUS SCREWS** 1134  
**FULLY THREADED HEAD - 3.5MM, 12MM TO 50MM**  
**LIFT STAR CANCELLOUS SCREWS** 0834  
**FULLY THREADED HEAD - 3.5MM, 12MM TO 50MM**



(DIFF UPTO 30MM, 2MM & FROM 30 ONWARD 5MM UPTO 85MM)

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 18 of 21

**LIFT INSTRUMENTS SET FOR 3.5 SCREWS**
**LIFT - DRILL SLEEVE 2.8 MM-S.S.**  
**LIFT - DRILL SLEEVE 2,8 MM-TIT,**

 CAT NO. 601.06  
 CAT NO. 3035

**LIFT - DRILL BIT 2.8 MM**  
**LIFT - DRILL BIT 2,5 MM**

 CAT NO. 601.14  
 CAT NO. 601.13

**LIFT - HEX. SCREW DRIVER 3,5 MM**

CAT NO. 601.15


**LIFT - DEPTH GAUGE 3,5 MM with Scale**

CAT NO. 601.34


**LIFT - K -WIRE 1,5 / 2,0 MM**

CAT NO. 1201


**LIFT - BONE TAP 3,5 MM**

CAT NO. 255.11


**LIFT - DRILL GUIDE 2.5 / 3.5 MM**

CAT NO. 601.05


**LIFT - STAR SCREW DRIVER 3,5 MM**

CAT NO. 601.27



Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 19 of 21



Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 20 of 21

**Yogeshwar Implants (I) Pvt. Ltd.**

Sr. No. 91 HISSA No. 1, "AR" building,  
Shobhana Complex, Rajlaxmi Compound,  
Kalher Village, Tal. Bhiwandi, Thane – 421302, INDIA

**E-mail:** sales@yogeshwarimplants.com

**Contact No:** +91-9323927102

**Website:** www.yogeshwarimplants.com

Doc. No.	Issue No/Rev No	Dated	Page No
YIPL/ST/24	01/01	27/01/2020	Page 21 of 21